

Patient Application Instructions

BEFORE USING THE DEVICE, PLEASE READ THE INSTRUCTIONS FOR USE COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE:

This brace is designed to alleviate pain by supporting the body in guiding and stabilizing the knee joint.

INDICATIONS:

For patients with symptoms, physical findings or radiographic evidence of moderate to severe unicompartmental knee osteoarthritis, or post-operative meniscal tears only where unicompartmental unloading is indicated.


APPLICATION OF THE BRACE:



1. Disconnect magnetic latches on thigh and calf



2. While sitting on the edge of a chair, flex the leg at 45 degrees, position hinge at top of the knee cap and slightly in front of midline of the leg on the affected compartment

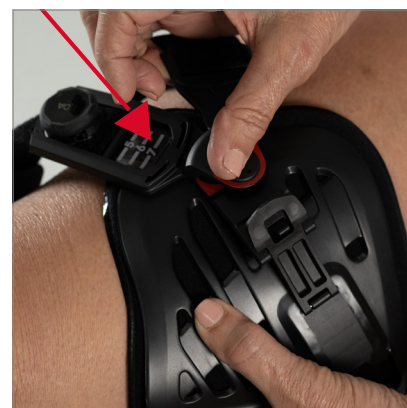

3. With buckles in the open position, connect the magnetic latches on thigh and calf



4. Bring thigh and calf buckles to closed position




5. Adjust thigh and calf circumferential straps as necessary



6. For thigh tensioning, move leg back into flexed 45 degree position, rotate BOA® dials clockwise to tighten as necessary, note location on printed graduated scale. Repeat for calf

ORDERING INFORMATION

PART NUMBER	DESCRIPTION	SIZE
11-1725-1-06000	ROAM OA, MDL LT/LAT RT	XS/S
11-1725-3-06000	ROAM OA, MDL LT/LAT RT	M/L
11-1725-5-06000	ROAM OA, MDL LT/LAT RT	XL/2XL/3XL
11-1726-1-06000	ROAM OA, MDL RT/LAT LT	XS/S
11-1726-3-06000	ROAM OA, MDL RT/LAT LT	M/L
11-1726-5-06000	ROAM OA, MDL RT/LAT LT	XL/2XL/3XL
11-R7222-0-00000	ROAM OA, SHTH, SHCA, MDL RT/LAT LT	-
11-R7223-0-00000	ROAM OA, SHTH, SHCA, MDL LT/LAT RT	-
11-R7242-0-00000	ROAM OA, SHTH, STCA, MDL RT/LAT LT	-
11-R7243-0-00000	ROAM OA, SHTH, STCA, MDL LT/LAT RT	-
11-R7422-0-00000	ROAM OA, STTH, SHCA, MDL LT/LAT RT	-
11-R7423-0-00000	ROAM OA, STTH, SHCA, MDL LT/LAT RT	-
11-R7442-0-00000	ROAM OA, STTH, STCA, MDL RT/LAT LT	-
11-R7443-0-00000	ROAM OA, STTH, STCA, MDL LT/LAT RT	-

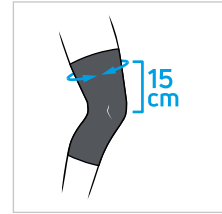

USE AND CARE:

This product is intended for single patient use only.

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners, straps and condyle pads in cold water with mild detergent. AIR DRY ONLY; DO NOT HEAT DRY LINERS, STRAPS OR PADS. Use wet wash cloth with mild detergent to hand wash uprights and shell. Rinse entire brace in cold water. AIR DRY.
- Lubricate hinges periodically with dry lubricant or Teflon™ spray.
- To lubricate internal hinge mechanism, place brace in flexion and spray into the back side of the hinge cover.
- This device is not intended to be used with other devices except those specifically listed as accessories to this device.
- CONTACT BRACE FITTER OR CLINICIAN for any brace modifications or if screws become loose.

Always consult with your physician or therapist before making changes to the brace.



WEBPAGE

Intended for use on a single patient.

Rx Only

LATEX FREE

enovis™

Creating Better Together™

enovis-medtech.eu

1a Guildford Business Park, Guildford, Surrey, GU2 8XG, UNITED KINGDOM

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.