## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER

The intended user should be the patient, a licensed healthcare professional, or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

### INTENDED USE/INDICATIONS

The Fusioglight II is designed to support the ankle for mild to moderate ankle sprains and help with chronic ankle instabilities (CAI).

- Acute lateral ligament injuries grade 1&2
- Conservative treatment of ligament injuries
- Initial management of chronic ankle instabilities
- Prevention of re-injury

### PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing mild protection of a limb or body segment for acute and prophylactic care.

### CONTRAINDICATIONS

- Do not use if you are allergic to any of the materials contained in this product
- Peripheral arterial diseases
- Impaired blood flow in the leg (e.g. with diabetes mellitus)
- Impaired lymphatic drainage or soft tissue swelling of unknown origin

### WARNING AND PRECAUTIONS

We recommend that the first application be made under supervision of a health care professional. If you have skin conditions/trauma in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional. If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while using this product, consult your health care professional immediately.

This product is for single patient use only.

Do not use this device if it is damaged and/or packaging has been opened.

The product should be removed during long periods of rest.

It is not recommended to drive while wearing the brace as your ability to brake may be impaired.

Do not use during showering or bathing.

Keep the product away from children.

Do not use on open wounds/s and/or sensitive skin.

Do not use this product if it is damaged or defective.

Do not use this product in any way that may damage its integrity.

Do not fit the Fusioglight too tightly, as this could lead to local pressure symptoms.

**NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.**

### APPLICATION INSTRUCTIONS

The product is designed to be worn during normal everyday activities and sports. It can be worn over a sock.

For additional support and optimal stabilization, we recommend to wear a closed shoe with the brace.

1. Place the liners pads inside the rigid bilateral shells.

2. Adjust the heel strap for proper fit.

3. Pull the outer shells and foam liners along the ankle, ensuring the malleoli are well protected.

4. Make sure the brace fits snugly, and then close the straps on the ankle and foot.

5. Adjust the heel pad with the vertical side straps while the brace is on if necessary. Pull straps upward until the desired fit is obtained.

6. For maximum comfort and support, wear a long absorbent sock and a laced athletic shoe when wearing the brace.

### CLEANING INSTRUCTIONS

Hand wash (max 30°C) with a mild detergent.

Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

### COMPOSITION

Brace composition: Polyurethane, Polyamide, Polypropylene, Acrylic resin

Pads composition: Polyester, Polyurethane, Polyamide, Acrylic resin

### INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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©2022 DJO, LLC 13-00072 REV A - 2022-08-02
5519 SAS Drive, Place
DJO France SAS
MDSS GmbH