

enovis™

DONJOY®

**LIFE'S NEXT CHAPTER
BEGINS WITH**

ROAM™ OA



DONJOY®

Life's next chapter begins with

ROAM™ OA

THE NEWEST INNOVATION IN UNLOADER BRACES FROM DONJOY®, PART OF ENOVIS™

Help patients say goodbye to knee pain and hello to ROAM OA™. From picking up their grandkids to teeing off at the driving range, **lightweight, low-profile joint unloading and support** never felt so good—or was so easy to prescribe. ROAM helps **improve mobility** and **provides relief** by unloading the pressure of unicompartmental osteoarthritis or other knee pain.

INTENDED USE/INDICATIONS

For patients with symptoms, physical findings or radiographic evidence of moderate to severe unicompartmental knee osteoarthritis, post-operative meniscal tears only where unicompartmental unloading is indicated, or varus-valgus knee conditions that may benefit from unicompartmental unloading.

COMFORTABLE, EASY TO USE AND EFFECTIVE

Ideal for a wide variety of patients who want to **increase activity** and who **enjoy an active lifestyle**. For conservative care or in preparation for surgery, ROAM is easy to use and offers a high level of comfort. Consistent use of an unloader knee brace has been shown to contribute to general daily functioning, reduced pain, and improved mobility, which can lead to **increased activity and greater quality of life**.



CUSTOM-FABRICATED OPTION FOR UNIQUE FIT NEEDS


A custom option addresses your most challenging fitting needs. Custom-measured with condyle and uprights custom-positioned and optimal height setting for the patient's "best fit," ROAM™ provides **low-profile, maximum comfort** and **mitigates migration** by capturing the calf muscle more effectively. Cuff sizes and strap lengths are tailored to the patient. Extension and flexion stops are pre-built into the brace to fulfill the patient's specific needs.

COMFORTABLE, EFFECTIVE AND SLEEK

- **Ultra-low profile, lightweight, fits under clothing**—all-black styling is modern, discreet and breathable
- **Unloads weight** from knee to **dynamically relieve pain**
- **Dynamic strapping and unique condyle harness** completely **avoids sensitive skin in the popliteal space**, while **providing comforting proprioception** on the opposite side of the knee
- **Flexible cuffs** comfortably cradle and **support thigh and calf**
- Designed for a **wide range of body types** and patient needs
- Soft silicone-enhanced lining **feels great on skin; prevents slippage**—2x Cool Technology fibers leave a lasting, cool feeling
- **Bendable uprights** on the OTS version allow you to better **match patient anatomy**

EVERY BRACE ALLOWS TWO FITMENT OPTIONS

Uniquely designed bracing that can be fitted for either **left lateral and right medial**, or vice versa fitting **right lateral and left medial**. Resulting in a drastically reduced SKU count. **Less is truly more.**

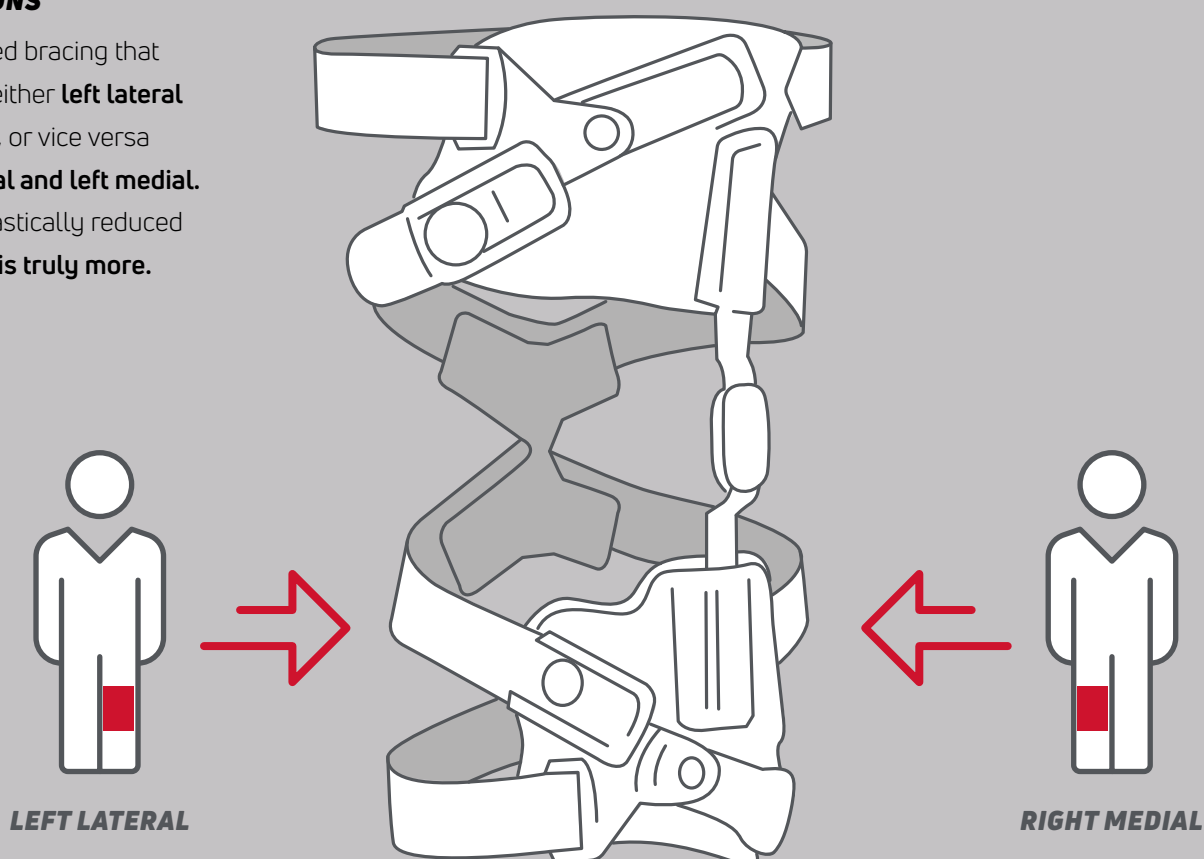



PATIENT-FOCUSED; PRESCRIBE FOR USE AND SATISFACTION

- “Set and forget” straps set by a trained fitter
- Two easy connection points; an audible “snap” of the **secure magnetic buckles** gives patients (including those with eyesight or dexterity challenges) **confidence and assurance**
- Patients can **dial in support and pain relief on demand**; BOA® Fit System makes it easy and clear with a visual indicator




DYNAMIC
UNLOADING


QUALITY & DURABILITY

- **Hand washable** after everyday outings and active adventures
- Backed by **DonJoy®** legacy of **quality, innovation, and clinically proven pain relief**
- Compliance—ROAM is **easy to wear**; put it on, take it off and adjust it in seconds
- Designed for efficiency, ease and a smaller environmental footprint; adjustability means just **a few core sizes** can be fit and customized to **most body shapes and sizes**
- **Only 6 off-the-shelf SKUs** are needed to cover the same range that traditionally requires over 50 SKUs


FEATURES AND BENEFITS

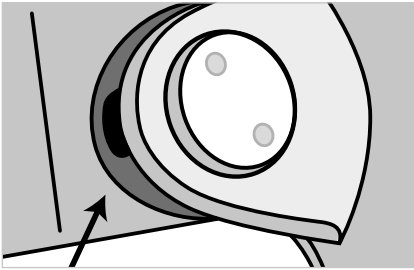


EASY MAG-ALIGN CLIPS: These patented magnetic clips help align the strap connections without any fuss. Just hold the end near the attachment cap and 'click', you're set.

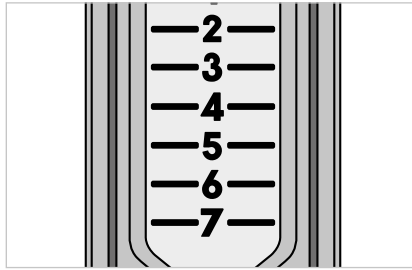


SET AND FORGET TECHNOLOGY: You can don and remove your brace without changing any of your doctor's settings.

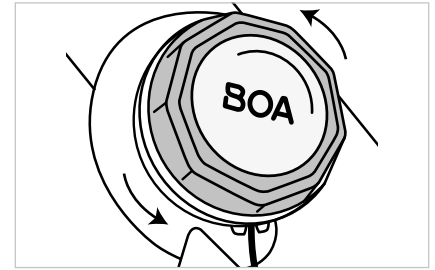

BOA®* DIAL FOR JUST THE RIGHT FIT: Dial up or dial down to set exactly how snug the brace fits.

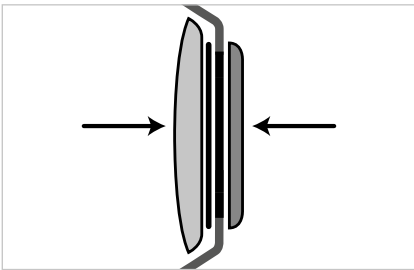

SLIM PROFILE HINGE: The slim hinge design allows the brace to be worn under relaxed clothing for a discrete fit.

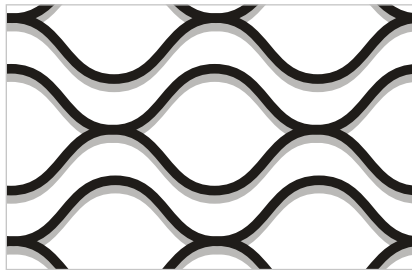

ANTI-MIGRATION TECHNOLOGY: Soft silicone helps keep the brace in place for steady comfort throughout the day.




PATENT-PENDING COMFORT STRAPS: The positioning of the support straps are engineered to help the back of your knee remain free from strap rub.

ORDERING INFORMATION

PART NUMBER		THIGH CIRC.	KNEE CENTER	CALF CIRC.	SIZE
RIGHT MEDIAL/ LEFT LATERAL	LEFT MEDIAL/ RIGHT LATERAL				
11-1726-1-06000	11-1725-1-06000	33 - 47 cm	30.5 - 35.5 cm	25.5 - 35.5 cm	XS/S
11-1726-3-06000	11-1725-3-06000	47 - 59.5 cm	35.5 - 43.2 cm	35.5 - 45.5 cm	M/L
11-1726-5-06000	11-1725-5-06000	59.5 - 81 cm	43.2 - 58.4 cm	45.5 - 61 cm	XL/XXL/XXXL

PART NUMBER	DESCRIPTION
11-R7222-0-00000	Short Thigh, Short Calf, Medial Right/Lateral Left
11-R7223-0-00000	Short Thigh, Short Calf, Medial Left/Lateral Right
11-R7242-0-00000	Short Thigh, Standard Calf, Medial Right/Lateral Left
11-R7243-0-00000	Short Thigh, Standard Calf, Medial Left/Lateral Right
11-R7422-0-00000	Standard Thigh, Short Calf, Medial Right/Lateral Left
11-R7423-0-00000	Standard Thigh, Short Calf, Medial Left/Lateral Right
11-R7442-0-00000	Standard Thigh, Standard Calf, Medial Right/Lateral Left
11-R7443-0-00000	Standard Thigh, Standard Calf, Medial Left/Lateral Right

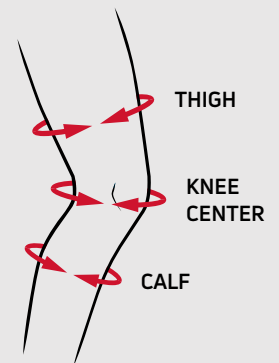

Scan code for a complete list of accessories and educational resources.



DONJOY®

Asia Pacific

1905, Tower 2, Grand Central Plaza
Shatin
HONG KONG
T: 852 31051415
E: marketing.ap@enovis.com

Australia – New Zealand

PO Box 6057
Frenchs Forest DC
NSW 2086
AUSTRALIA
T: 1300 66 77 30
E: customerservice.au@enovis.com

Benelux

Kleinhoefstraat 5, bus 39
Geel 2440
BELGIUM
T: +32 (0)14 248090
E: orders.bx@enovis.com

Canada

6485 Kennedy Rd
Mississauga
Ontario, L5T2W4
CANADA
T: 1-866-866-5031
E: canada.orders@enovis.com

China

A312 SOHO ZhongShan Plaza
1055 W. ZhongShan Rd
Shanghai, 200051
CHINA
T: (8621) 6031 9989
E: shujun.tang@enovis.com

France

3 rue de Bethar
Centre Européen de Frêt
64990 Mouguerre
FRANCE
T: +33 (0)5 59 52 86 90
E: sce.cial@enovis.com

Germany

Bötzingen Straße 90
79111 Freiburg
GERMANY
T: +49 (0) 180 1 676 333
E: kundenservice@enovis.com

Italy

Via Leonardo da Vinci 97
Trezzano sul Naviglio
Milan 20090
ITALY
T: +39 02 835 98001
E: vendite@enovis.com

Nordic

Murmansgatan 126
212 25 Malmö
SWEDEN
T: +46 40 39 40 00
E: info.nordic@enovis.com

South Africa

Unit 6, 2 on London, 2 London Circle
BrackenGate Business Park
Brackenfell, 7560
SOUTH AFRICA
T: +27 (0) 21 276 2968
E: info.southafrica@enovis.com

Spain

Av.Cornella 144. 1-4
08950 Esplugues de Llobregat
Barcelona
SPAIN
T: +34 93 480 32 02
E: ventas@enovis.com

UK & Ireland

1a Guildford Business Park
Guildford, Surrey
GU2 8XG
UK
T: +44 (0)1483 459 659
E: ukorders@enovis.com

USA

2900 Lake Vista Drive
Lewisville
TX 75067
U.S.A.
T: 760 727 1280
E: customercare@enovis.com

enovisTM

Creating Better Together™

enovis-medtech.eu

1a Guildford Business Park, Guildford, Surrey, GU2 8XG, UNITED KINGDOM

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.

*BOA® is a registered trademark of Boa Technology, Inc.