

AIRCAST®

Hallux Valgus
& Foot Protection Range

Rediscover the joy
of a carefree step.



 DJO®

AIRCAST®

Tread carefully with toe deformity - it's a slippery slope.

Hallux Valgus - or bunion - is one of the most common foot deformities, especially for women. Because of its weaker connective tissue, the big toe can become misaligned, resulting in an unsightly and painful protrusion from the foot.

Hallux Valgus usually develops as a result of years of splayfoot, the most common painful malformation of the foot. This causes the first and second metatarsal bones to spread apart, leading to incorrect loading of the foot. The misalignment of the big toe often increases over time due to the changed stress caused by the splayfoot and the changed tendon pull. In severe cases, the misalignment of the big toe can also affect the adjacent foot skeleton and lead to a misalignment of the second and third toes.

As well as its unattractive appearance, toe deformity leads to problems when choosing a shoe, and also causes painful pressure points and inflammation. Without corrective action, all of this can lead to a loss of confidence and a reduction in daily activity.



Introducing the new standard in foot care.

For generations, our Aircast® products have provided patients with innovative healthcare solutions. Now it's time for us to set a new standard in foot care with the introduction of this comprehensive portfolio of Hallux Valgus and foot protection products.

Designed to address a variety of foot disorders, together they help provide patients with correction, protection and pain relief.

Say hello to this innovative product family and rediscover the joy of a carefree step.





ActyToe™

Hinged splint to help bunion pain relief and to aid correction, for use during normal daily activity, during rest and throughout the night.

Intended use/Indications

- Pain relief in patients with symptomatic Hallux Valgus (bunion)
- Mild or moderate mal-positioning of the big toe
- Maintaining position after Hallux Valgus (bunion) surgery

Characteristics

- Low-profile design
- Locking clip for splint immobilization (0° and 30° dorsi- and plantar flexion)
- Memory foam cushion
- Adjustable correction angle (0° and 10°)
- Adjustable, thin and soft straps
- Anatomically contoured splint
- Breathable and lightweight materials
- For day and night wear
- Suitable for walking
- Available in 3 sizes (S, M, L)
- For left and right foot
- Can be worn with wide footwear
- Latex free

Ordering Information

Part Number	Shoe size			Size
	EU	UK Men	UK Women	
82-04S	< 36	< 4	< 3.5	S
82-04M	35 - 42	3.5 - 8	3 - 7.5	M
82-04L	41 - 46	7.5 - 11.5	7 - 11	L

Features and benefits

Available in 3 sizes

For optimal fit

Adjustable mid-foot strap


To support the transverse arch and help stabilize the mid-foot

Articulated splint

Helps achieve pain-free movement of the big toe

Adjustable big toe strap

To help gently correct misalignment of the big toe

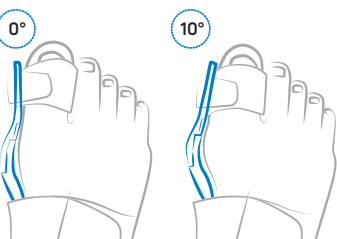


Interchangeable big toe part

- Offers gentle correction at 2 angles (0° or 10°) to aid different degrees of deformation
- For individualized fit and increased patient comfort and compliance

Soft "memory foam" cushion

- To comfort and help protect the bunion
- Aid relief of the metatarsal-phalangeal joint against pressure points

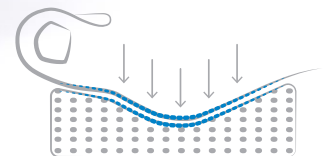

Lockable joint in 0° or 30° dorsi- and plantar flexion

Customized application: three optional locking positions to hold the big toe at a predetermined angle, depending on treatment requirements



ActyToe™ Night

Rigid splint to help bunion pain relief and to aid correction, for use during rest and throughout the night.



Intended use/Indications

- Pain relief in patients with symptomatic Hallux Valgus (bunion)
- Mild or moderate mal-positioning of the big toe
- Maintaining position after Hallux Valgus (bunion) surgery

Characteristics

- Low-profile design
- Memory foam cushion
- Adjustable, thin and soft straps
- Anatomically contoured splint
- Breathable and lightweight materials
- For night and rest periods
- Available in 3 sizes (S, M, L)
- For left and right foot
- Latex free

Ordering Information

Part Number	Shoe size			Size
	EU	UK Men	UK Women	
82-05S	< 36	< 4	< 3.5	S
82-05M	35 - 42	3.5 - 8	3 - 7.5	M
82-05L	41 - 46	7.5 - 11.5	7 - 11	L

Features and benefits

Adjustable mid-foot strap

To support the transverse arch and help stabilize the mid-foot

Available in 3 sizes

For optimal fit

Adjustable big toe strap

To help gently correct misalignment of the big toe




Rigid splint

For immobilization and support at night and during rest

Soft "memory foam" cushion

- To comfort and help protect the bunion
- Aid relief of the metatarsal-phalangeal joint against pressure points

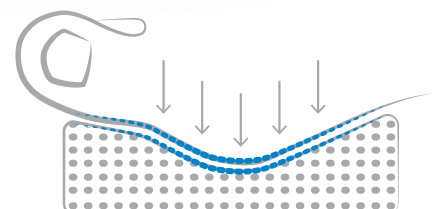



ActyToe™ Lift

Mid-foot brace with pad for bunion and splay foot pain relief and transverse arch support.

Intended use/Indications

The mid-foot brace is designed to lift the mid-foot bone and support transverse arch, helping provide pain relief for conditions such as:

- Hallux Valgus
- Splay foot, flat foot (fallen arch)
- Metatarsalgia

Characteristics

- Low-profile design
- Adjustable strap with hook and loop fastener
- Cushioning arch support pad
- Universal size
- For left and right foot
- Can be worn in a shoe
- Latex free

Ordering Information

Part Number	Size
82-06	Universal

Features and benefits

Adjustable strap

- Provides a strong hold and effective support to aid in pain relief
- For easy application and adjustment

Thin and soft materials

Help to maximize patient comfort and compliance

Low-profile design

Fits comfortably under socks and in shoes

Foot arch support pad

Lifts the transverse arch and stabilizes the mid-foot bones to relieve pressure



SofToes™

Rediscover the joy of movement.

The innovative Aircast® SofToes™ range provides a perfectly fitting solution for every need in the toe area. These ergonomic cushions help provide gentle protection, soft comfort and secure support, while a subtle hint of menthol helps maintain a fresh, airy smell.

General features and benefits

- Light menthol scent for a fresh smell
- Soft touch for greater comfort
- Durable, easy to clean and reusable
- Lightweight, skin-friendly materials, hypoallergenic, PVC and latex free
- Supplied coated in talcum powder for initial ease of application and comfort
- One size fits most



Toe cushion

Intended use/Indications

- Helps to relieve pain in the toe area
- Helps to reduce pressure on the toes and metatarsal heads
- Helps to maintain the toes in good position

Ordering Information

Part number	Contents	Size
82-10-01	1 pair (left and right)	Universal



Forefoot pad

Intended use/Indications

- Helps to protect and relieve pressure on the forefoot area
- Can also serve as a non-slip foothold

Ordering Information

Part number	Contents	Size
82-10-02	1 pair (left and right)	Universal



Forefoot pad with ring

Intended use/Indications

- Helps to protect and relieve pressure on the forefoot area
- Provides support of the transverse arch, cushioning and comfort

Ordering Information

Part number	Contents	Size
82-10-03	1 pair (left and right)	Universal



Toe separator

Intended use/Indications

- Reduces friction and pressure between the toes and helps to prevent blisters, corns and rubbing nails
- Helps to separate dislocated and closely positioned toes
- Helps to prevent pain from pressure or rubbing, skin irritation and corns between the toes

Ordering Information

Part number	Contents	Size
82-10-04	2 pieces	Universal



Toe cap

Intended use/Indications

- Helps to prevent pain from pressure points
- Helps to relieve pressure and friction

Ordering Information

Part number	Contents	Size
82-10-05	2 pieces	Universal



Toe spreader

Intended use/Indications

- Separates dislocated and closely positioned or overlapping big and second toes
- Helps to prevent pain from pressure or rubbing, skin irritation and corns between the big and second toes
- Helps to relieve pain and discomfort from bunions
- Helps to limit the deviation of the big toe (Hallux Valgus) by improving its alignment

Ordering Information

Part number	Contents	Size
82-10-06	2 pieces	Universal



Little toe protector

Intended use/Indications

- Helps to relieve pain in case of ball problems at the little toe (little toe bunion/bunionette/tailor bunion)
- Helps to protect against further pressure, friction and irritation
- Helps to prevent corns and calluses, and sore skin and swelling

Ordering Information

Part number	Contents	Size
82-10-07	1 piece	Universal



Toe ring

Intended use/Indications

- Helps to prevent pain from pressure points
- Helps to relieve pressure and friction
- Helps to prevent corn formation

Ordering Information

Part number	Contents	Size
82-10-08	2 pieces	Universal



Bunion protector

Intended use/Indications

- Separates dislocated and closely positioned or overlapping big and second toes
- Helps to prevent pain from pressure or rubbing, skin irritation and corns between the big and second toes
- Helps to limit the deviation of the big toe (Hallux Valgus) by improving its alignment
- The ball protector provides pain and pressure relief in case of ball soreness

Ordering Information

Part number	Contents	Size
82-10-09	1 piece	Universal

AIRCAST®

DJO Asia Pacific

1905, Tower 2, Grand Central Plaza
Shatin
HONG KONG
T: 852 31051415
E: marketing.ap@djoglobal.com

DJO Australia

ANZ Asia
PO Box 6057
Frenchs Forest DC
NSW 2086
AUSTRALIA
T: 1300 66 77 30
E: customerservice.au@djoglobal.com

DJO Benelux

Kleinhoefstraat 5, bus 39
Geel 2440
BELGIUM
T: +32 (0)14 248090
E: benelux.orders@djoglobal.com

DJO Canada

6485 Kennedy Rd
Mississauga
Ontario, L5T2W4
CANADA
T: 1-866-866-5031
E: canada.orders@djoglobal.com

DJO China

A312 SOHO ZhongShan Plaza
1055 W. ZhongShan Rd
Shanghai, 200051
CHINA
T: (8621) 6031 9989
E: information_china@djoglobal.com

DJO France

3 rue de Bethar
Centre Européen de Frêt
64990 Mouguerre
FRANCE
T: +33 (0)5 59 52 86 90
E: sce.cial@djoglobal.com

DJO Germany

Bötzingen Straße 90
79111 Freiburg
GERMANY
T: +49 (0) 180 1 676 333
E: kundenservice@djoglobal.de

DJO Italy

Via Leonardo da Vinci 97
Trezzano sul Naviglio
Milan 20090
ITALY
T: +39 02 835 98001
E: vendite@djoglobal.com

DJO Nordic

Murmansgatan 126
212 25 Malmö
SWEDEN
T: +46 40 39 40 00
E: info.nordic@djoglobal.com

DJO South Africa

Unit 6, 2 on London, 2 London Circle
BrackenGate Business Park
Brackenfell, 7560
SOUTH AFRICA
T: +27 (0) 21 276 2968
E: info.southafrica@djoglobal.com

DJO Spain

Av.Cornella 144. 1-4
08950 Esplugues de Llobregat
Barcelona
SPAIN
T: +34 93 480 32 02
E: ventas@djoglobal.com

DJO UK & Ireland

1a Guildford Business Park
Guildford, Surrey
GU2 8XG
UK
T: +44 (0)1483 459 659
E: ukorders@djoglobal.com

DJO USA

2900 Lake Vista Drive
Lewisville
TX 75067
U.S.A.
T: 760 727 1280
E: customercare@djoglobal.com



AIRCAST®

DJO Global | 1a Guildford Business Park | Guildford | Surrey | GU2 8XG | UNITED KINGDOM

www.djoglobal.eu

Individual results may vary. Neither DJO Global, Inc. nor any of its subsidiaries dispense medical advice. The contents of this brochure do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.